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# ADDITIONAL PRODUCT INSERT FOR OPTICIANS AND CONTACT LENS WEARERS

## IMPORTANT INFORMATION for the fitting of contact lenses:

Due to the fact that not all materials were trialled with all the different parameters and geometries in clinical studies with a significant number of patients, the optician must take into account all the characteristics of the lenses that have an influence on the performance of the contact lens and the health of the eye, including permeability to oxygen, moisture, interior and exterior strengths as well as strengths in the optical field of vision.

The potential impact of these factors on the health of the eye must be carefully weighed up against the correction to the eyesight with a contact lens. The optician should thus carefully monitor the health of the eye as well as the performance of the contact lens on a regular basis.

Patients who wear aspheric contact lenses for the correction of long-sightedness (hyperopia) are possibly not best served when it comes to clarity of vision at close and long range. Vision requirements vary for every patient on an individual basis and this should be taken into account in the selection of the appropriate contact lens.

The dye Fluorescein should not be used if the contact lens is still being worn. Following the application of Fluorescein to the eye, the eyes must be rinsed well with a sterile saline solution which can be applied to the eye in the form of drops. Patients must be informed that they may only insert their contact lenses one hour after this application.

Following the initial fitting of the contact lenses, the wearer should be able to independently remove them before he/she leaves the optician's or there should be someone with him/her who can assist with this.

### 1. Contraindications

In the event of the occurrence of the following contraindications, the wearing of contact lenses is no longer appropriate and/or not appropriate for a longer period by the patient

- 1.1 Known, abnormal, recurrent reactions of the anterior eye segment to wearing contact lenses.
- 1.2 Diseases, injuries or changes to the anterior eye segment (cornea, conjunctiva, eyelids) such as corneal neovascularisation, corneal erosion, corneal ulcer, in the case of viral, fungal or bacterial infections as well as in the case of acute or subacute inflammatory processes in the eye
- 1.3 In the case of reduced corneal sensitivity
- 1.4 In the case of dry eyes (insufficient tear film)
- 1.5 In the case of systemic diseases that affect the eye or diseases that are exacerbated by the wearing of contact lenses
- 1.6 In the case of allergies to the ingredients listed in the packaging for the contact lenses or in the contact lens care packaging
- 1.7 If it is not possible for the wearer of the contact lenses to implement the instructions for use himself/herself and it is not possible for him/her to be given assistance with this.

Opticians must instruct patients very clearly in the care of extended-wear contact lenses and carefully explain the following warnings and safety measures that are listed in the instructions for use:

Patients who are wearing contact lenses for the first time must be informed that they should only be worn when the patient is awake and only for 6 hours at a stretch, with a daily increase in the wearing time of one hour per day, up to a maximum of 12 hours per day while awake.

### 2. Warnings

- 2.1 Studies show that the risk of a serious corneal inflammation is increased in the case of extended daily wearing times or the permanent wearing of contact lenses.
- 2.2 Studies show that smokers have an increased risk of effects that have a negative impact on the wearing of contact lenses in comparison with non-smokers
- 2.3 After eye operations or illness, contact lenses may only be worn again after medical approval has been granted.
- 2.4 If the seal on the contact lens packaging or the contact lens should be damaged, the contact lens must not be worn. Dehydrated (dry) contact lenses must not be used and must be disposed of immediately.

2.5 In order to ensure the health of the eye, it is important that the contact lenses can move easily on the eye. If your contact lenses sit rigidly on the eye, please consult your optician.

2.6 The UV protection of a contact lens does not remove the need for sunglasses.

2.7 Remove the contact lens immediately in the event of discomfort, redness of the eyes, increased tear formation, a foreign body sensation, itching, pain, blurred vision, sensitivity to light or halos around lights. If this discomfort is not spontaneously resolved by cleaning, remove the contact lenses and immediately consult your optician or an ophthalmologist.

### Precautions that must be observed by the wearer of the contact lenses:

1. No tap water, saliva or other liquid other than the contact lens care system may be used to rinse the contact lenses or their containers. This avoids any increase in the risk of serious microbial infection.
2. Only the disinfection and care system recommended by the optician may be used, and the instructions for use for the care- system must be observed.
3. Consult your optician before you start to wear the contact lenses again after a long period of not doing so.
4. If you do not wear your contact lenses on a daily basis, they must be cleaned and disinfected in accordance with the method of disinfection and care used before they are worn again. This is to avoid infection. Otherwise the contact lenses must be discarded. The instructions for using the method of disinfection and care must be observed.
5. The replacement schedule for your contact lenses must be observed. The replacement schedule (daily, weekly, monthly) is shown on the packaging.
6. Some medications may interfere with the wearing of contact lenses. In order to avoid this, please inform your optician of any changes to your medication or new medication used.
7. Do not expose the contact lenses to hairspray, cosmetics or aerosol spray; this also applies to working in areas contaminated with aerosol spray or chemical vapours. If a chemical substance should come into contact with the eye, the contact lens must be removed immediately. The eye must be rinsed and a doctor consulted without delay.
8. Contact lenses must not come into contact with bath water or swimming pool water in order to avoid the loss or contamination of the contact lenses and to avoid any serious microbial infections. Goggles can prevent contact lenses from coming into contact with the water.
9. Contact lenses should only be worn during sleep following prior consultation with your optician.
10. Doctors and, where necessary, employers should be informed that you wear contact lenses.
11. Do not exceed the wearing time recommended by your optician.
12. The recommended follow-up checks must be observed in order to identify any ocular changes and to adjust the recommendations for use accordingly.